**NE1001**

ANNI DI **GARANZIA**
YEARS **GUARANTEE**
ANOS DE **GARANTIA**
ANOS DE **GARANTIA**
JAHRE **GARANTIE**
ANS DE **GARANTIE**
XPONIA **ΕΓΓΥΗΣΗ**
ANI DE **GARANTIE**
ROK **ZÁRUKA**
ROK **ZÁRUKA**
ÉV **GARANCIA**

Data - Date

--	--	--	--	--	--	--	--

Timbro rivenditore
Retailer's stamp
Sello del revendedor
Carimbo do revendedor
Stempel des Händlers
Cachet du revendeur
Σφραγίδα καταστήματος
Ștampilă distribuitor
Razítko prodejece
Pečiatka predajcu
Boltí pecsét

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ULTRASONIC AEROSOL THERAPY DEVICE INSTRUCTIONS AND WARRANTY

Dear Customer, Laica would like to thank you for the preference accorded to this product, designed according to the criteria of reliability and quality in order to meet your complete satisfaction.

IMPORTANT PLEASE READ CAREFULLY BEFORE USE KEEP FOR FUTURE REFERENCE

The instruction manual must be considered an integral part of the product and must be kept for its entire life. If the appliance is transferred to another owner, its documentation must also be transferred in its entirety. To ensure the safe and correct use of the product, the user must carefully read the instructions and warnings contained in the manual insofar as they provide important information regarding safety, user and maintenance instructions.

In the event of loss of the instruction manual or the need to receive more information or clarification, please contact us at the address below:

LAICA S.p.A. - Viale del Lavoro, 10 - 36048 Barbarano Mossano (VI) - Italy - Phone +39 0444.795314 - info@laica.com - www.laica.it

This aerosol is an effective tool for home treatment of disorders of the upper and lower respiratory tract. It is compact, quiet, easy to use, built in compliance with the current European regulations regarding design criteria for the safety of the electrical appliances for medical use (Dir.93/42/EEC and Standard EN13544-1).

KEY TO SYMBOLS

Warning

Prohibition

CE0425

In compliance with European legislation on medical devices

Batch production number

Manufacturer

Caution! Read the user instructions carefully

Symbol for "Type BF applied parts"

Class II appliance

IP21: Degree of protection of electrical equipment covers, where the first figure indicates the degree of protection against penetration by solid foreign bodies (from 0 to 6) and the second figure the degree of protection against penetration by liquids (from 0 to 8).

SAFETY WARNINGS

- Prior to use, ensure the product is intact and that there is no visible damage. If in doubt, contact the retailer.
- Keep the plastic bag away from children: danger of suffocation.
- This device must be used only for its intended purpose and according to the user instructions. All other types of use shall be considered inappropriate and therefore hazardous. The manufacturer shall not be deemed liable for any damage caused by inappropriate or incorrect use.
- People aged 14 years or older, people with physical, sensory or mental disabilities or unskilled people, may use and maintain the product only under adult supervision. Children must not play with the device.
- **Keep all parts of the packaging as well as the air hose and adaptor away from children (risk of suffocation and strangling)**
 - Before connecting the device, make sure the mains voltage corresponds to the voltage on the plate affixed to the product.
- NEVER leave the device operating unsupervised. Once finished using, turn it off and unplug it from the mains.
- Handle the product with care, protect it against accidental impact, extreme temperature fluctuations, humidity, dust, direct sunlight and sources of heat.
- In the event of a failure and/or malfunction, turn off the device without tampering with it. Always contact the retailer for repairs.
- Be sure to have dry hands before connecting or disconnecting the device and when pressing the "ON" switch.

- ⊘ DO NOT pull the power cord to unplug the adapter from the power outlet.
- ⊘ DO NOT use extensions, unwind the power cord and keep it away from sources of heat.
- ⊘ DO NOT tamper with the power cord for any reason. If damaged, contact your retailer.
 - Always use the "ON" switch to turn the device off and then unplug the adapter.
- Unplug the power cord and the adapter immediately after use and before cleaning.
- ⊘ DO NOT immerse the device and DO NOT pour water or other liquids onto it.
 - If the device falls in the water, do not try to pick it up. Immediately unplug it. DO NOT reuse the device after removing it from the water.
- Only use original accessories of the manufacturer.
- Before each use, make sure that the parts have been assembled correctly.
- ⊘ DO NOT use the device if the air tube is bent.
- Make sure that the air filter is inserted correctly and that it is clean.
- ⊘ Do not use the device close to high-frequency electromagnetic emitters.
- Keep the aeration opening free from dust and impurities. Avoid leaning the device on soft surfaces (i.e. bed or pillows).

IMPORTANT NOTES FOR BEST OPERATION

- 1) Do not use the equipment if the nebuliser chamber is without water or if the water level is below the indicated level.
- 2) The disposal medical cup is **SINGLE USE**; replace it after each use to guarantee best nebulisation and to avoid damage to the equipment.
- 3) Do not use oily medicines with a viscosity over 3 cPs (centipoise), the unit to measure viscosity, for example cortisone, as it could damage the transducer.
- 4) Do not use essential oils (such as eucalyptus essence), active ingredients with solid elements (suspended) and viscous solutions as they could damage the transducer.
- 5) **After every use**, the nebuliser chamber (no.8 exploded view) and the transducer (no.9 exploded view) should be cleaned with a cloth dipped in pure alcohol.
- 6) Wait at least 15 minutes from one use to the next.
- 7) Clean the top cover (no.5 exploded view) of the device handling the flow duct with care (no.6 exploded view).

ACCESSORIES/COMPONENTS

- 1) Main unit
- 2) "ON" key
- 3) Operation warning light ("ON/OFF")
- 4) Water tank
- 5) Upper cover
- 6) Stream conveyor
- 7) Disposable medicine cup (8cc) with spare parts
- 8) Nebulization chamber
- 9) Transducer
- 10) Flow control device (-/+)
- 11) Air connection pipe
- 12) Masks connector
- 13) Mask for adults
- 14) Mask for children
- 15) Mouthpiece
- 16) Adaptor plug
- 17) Adaptor

TECHNICAL SPECIFICATIONS

- **Power supply:**
 - INPUT: 110-240V ~ 50-60Hz, 0.4A;
 - OUTPUT: 13.5V, 1A
- **Operating conditions:**
 - Temperature: min. 5°C; max. 40°C
 - Relative humidity: 15-93%, in absence of condensate
 - Atmospheric pressure from 700 to 1060hPa
- **Storing conditions:**
 - Temperature: -25°C ~ +70°C
 - Relative humidity: less than 93%, in absence of condensate
- Ultrasound frequency: 1.7 MHz
- Nebulization capacity: about 0.4 ml/minute
- Percentage of aerosol of the inhalation: 60% of the particles < 5 µm
- Medium size of the particles: 3.4 µm
- Maximum volume of the inhaling product: max. 8 ml = 8 cc

INSTRUCTIONS FOR USE

- Before each use, carefully clean and disinfect the water tank, the cover, the masks connectors, the air connection tube, the mouthpiece and the masks according to paragraph "Cleaning and maintenance".
- Place the device on a flat surface.
- Remove the cover. The tank holds 15 cc. Fill with tap water up to the red level indicator. This water is used as liquid conducting to the drug the ultrasonic waves and is never nebulized. Do not put the tank directly under the tap to fill it with water. It is strictly forbidden to use salt solution and water with any chemical element.
Do not pour drugs directly in the tank.
- Fill the nebulizer with the drug, then place it in the water tank according to figure 3. Pour the drug paying attention not to exceed the maximum capacity (8cc). Follow the instructions for use and the dosing of the medicinal product you are using.
- Close the tank with the proper cover.
- Now, insert the pipe and the mouthpiece or the connector and the mask for adults/children in the cover, according to figure 6.
In case of inhalation through mouthpiece:
In case of troubles at low respiratory system, for example cough or bronchial infections, use the mouthpiece.
 - Sit upright.
 - Connect the air connection tube in the cover and in the other end insert the mouthpiece.
 - Tighten the mouthpiece well with the lips.
 - Inhale slowly and deeply through the mouth and exhale through the nose. To increase the efficacy of the therapy, hold your breath for a while after having inhaled.
 During the treatment, remain quiet and relaxed. Breathe regularly and not too quickly. If you wish to interrupt the inhalation shortly, remove the mouthpiece from the mouth and then reinsert it.

In case of inhalation with mask:
The mask for inhalations is suitable for the treatment of the nasal and pharyngeal area.
 - Sit upright.
 - Connect the air connection tube in the cover and in the other end insert the fitting and the mask.
 - Keep the mask lightly pressed against the nose; the mask has to cover mouth and nose without squashing them.
 - Inhale and exhale slowly and deeply through the nose. To increase the efficacy of the therapy, hold your breath for a while after having inhaled.
 During the treatment, remain quiet and relaxed. Breathe regularly and not too quickly. If you wish to interrupt the inhalation shortly, remove the mask for a while and then place it again on nose and mouth.
- Connect the transformer to the device and then to the power supply plug paying attention that the voltage is right.
- Switch on the device pressing the "⏻" key. Make sure that the warning light switches on (blue colour). During inhalation, the device must compulsorily remain in vertical position: this assures its correct operation, as well as the maximum efficacy of therapy. It is possible to control the speed the pharmaceutical is nebulized by turning the flow control (-/+). Immediately interrupt the treatment if you feel unease during the aerosol-therapy.
- The device has an automatic stop function, which deactivates the device when the medicine is almost exhausted or after about 15 minutes (+/- 2 min.) of inhalation. The device can be switched off at any time by pressing the "⏻" switch.
IMPORTANT: As with most equipment for aerosol therapy, at the end of the inhalation treatment, a certain amount of the pharmaceutical will remain in the nebulization ampoule: this is perfectly normal. This amount of pharmaceutical, also known as residual volume, cannot be nebulized.
- Let the device cool down for some minutes, turn the device upside down or use an absorbing cloth to remove the residual liquid and perform the cleaning procedures according to paragraph "Cleaning and maintenance".
- Wait at least 15 minutes from one use to the next.

REPLACEMENT OF THE PHARMACEUTICAL AMPOULE

The disposal medicine cup is **SINGLE USE**; replace it after each use to guarantee best nebulisation and to avoid damage to the equipment. Within the pack you will find ampoules which you can use for replacement purposes. Other ampoules can be purchased from your retailer.

MICROBIC CONTAMINATION

In the presence of conditions carrying the risk of microbial infection and contamination, personal use of the accessories only is advised (always consult a doctor).

CLEANING

- Caution! Hot surface!**
Before cleaning the device, let it cool since the transducer heats during the use.
- After having disconnected the socket, remove the cover and empty the tank from any eventual residuals.

- After every use**, the nebuliser chamber (no.8) and the transducer (no.9) should be cleaned with a cloth dipped in pure alcohol. Clean the top cover (no.5) of the device with a damp or vinegar-dipped cloth to remove all residues. During cleaning, handle the flow duct with care. (no.6)
- Clean the external part of the device using a soft cloth soaked with water and neutral soap (**do not clean the main unit by placing it under running water and pay attention that no liquid penetrates into the device**).
- In case of daily use, it is suggested to disinfect or sterilize the accessories all the days to limit the risk of infections. In this case, place all accessories, except for the air connection tube, in boiling water for 10 minutes and then let them dry completely before reusing them. Wash air connection tube carefully with drinking water. Leave to dry naturally.
- Disinfect or sterilize all components, except for the air connection tube, in boiling water for 10 minutes also before using the device for the first time. To disinfect the air connection tube it is possible to use 70% alcohol (or an alcohol-based spray disinfectant). Completely remove any residue of the disinfectant by rinsing the tube in abundant running water and let it air dry.
- To disinfect the device, it is also possible to use 70% alcohol (or a disinfecting spray with alcoholic base). Check that within the device there are not residual parts of the disinfectant.
- The device, the tank and all accessories must be completely dry before using them.
- Store the device in a fresh and dry place.

DISPOSAL PROCEDURE (Dir. 2012/19/Eu-WEEE)

The symbol on the bottom of the device indicates the separated collection of electric and electronic equipment. At the end of life of the device, do not dispose it as mixed solid municipal waste, but dispose it referring to a specific collection centre located in your area or returning it to the distributor, when buying a new device of the same type to be used with the same functions.

If the appliance to be disposed of is less than 25 cm, it can be returned to a retail location that is over 400 m² without having to purchase a new, similar device. This procedure of separated collection of electric and electronic devices is carried out forecasting a European environmental policy aiming at safeguarding, protecting and improving environment quality, as well as avoiding potential effects on human health due to the presence of hazardous substances in such equipment or to an improper use of the same or of parts of the same.

Caution! The wrong disposal of electric and electronic equipment may involve sanctions.

WARRANTY

This equipment is covered by a 2-year warranty starting from the purchase date, which has to be proven by means **of the stamp or signature of the dealer and by the fiscal payment receipt that shall be kept enclosed to this document**. This warranty period complies with the laws in force and applies only in case the consumer is an individual. Laica products are designed for domestic use only; therefore, their use for commercial purposes is forbidden.

The warranty covers only production defects and does not apply, if the damage is caused by accidental events, misuse, negligence or improper use of the product. Use the supplied accessories only; the use of other accessories invalidate the warranty cover. Do not open the equipment for any reason whatsoever; the opening or the tampering of the equipment, definitely invalidate the warranty. Warranty does not apply to parts subject to tear and wear along the use, as well as to batteries, when they are included. Warranty decays after 2 years from the purchase date. In this case, technical service interventions will be carried out against payment. Information on technical service interventions, under warranty terms or against payment, may be requested by writing at info@laica.com.

Repair or replacement of products covered by the warranty terms are free of charge. In case of faults, refer to the dealer; DO NOT send anything directly to LAICA. All interventions covered by warranty (including product replacement or the replacement of any of its parts) shall not be meant to extend the duration of the original warranty period of the replaced product.

The manufacturer cannot be made liable for any damages that may be caused, directly or indirectly, to people, things and pets resulting from the inobservance of the provisions contained in the suitable user manual, especially with reference to the instructions related to installation, use and maintenance of the equipment. Being constantly involved in improving its own products, Laica reserves the right to modify its product, totally or partially, with reference to any production need, without that this involves any responsibility whatsoever by Laica or its dealers.

- The device needs special precautions regarding electromagnetic compatibility (EMC) and needs to be installed and put into service according to the EMC information provided in the ACCOMPANYING DOCUMENTS;
- Portable and mobile RF communications equipment can affect the device.
- Warning that the use of accessories, transducers and cables other than those specified with the exception of transducers and cables sold by the manufacturer of the device as replacement parts for internal components, may result in increased EMISSIONS or decreased IMMUNITY of the device.
- Warning that the device should not be used adjacent to or stacked with other equipment.
During Voltage Dips test, the product will change its operation state from on to off, after test, it needs operator to press on/off button to recover. This phenomenon does not affect BASIC SAFETY or ESSENTIAL PERFORMANCE, according the standard, it's deemed to comply with the requirement.

Electromagnetic compatibility		
The DEVICE is intended for use in the electromagnetic environment specified below. The customer or the user of the DEVICE should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The DEVICE uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 14	Class B Conforms	It is possible to use the device in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings user for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class C Conforms	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

Immunity aspects			
The DEVICE is intended for use in the electromagnetic environment specified below. The customer or the user of the DEVICE should assure that it is used in such an environment.			
Immunity test	Test level EN 60601-1-2	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) EN 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Burst/Fast Transient EN 61000-4-4	±2kV power supply lines	±2kV power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge EN 61000-4-5	±1kV differential mode ±2kV common mode	±1kV differential mode ±2kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines EN 61000-4-11	< 5% UT (>95% dip in UT) for 0,5 cycles 40% UT (60% dip in UT) for 5 cycles 70% UT (30% dip in UT) for 25 cycles < 5% UT (>95% dip in UT) for 5 seconds	< 5% UT (>95% dip in UT) for 0,5 cycles 40% UT (60% dip in UT) for 5 cycles 70% UT (30% dip in UT) for 25 cycles < 5% UT (>95% dip in UT) for 5 seconds	Mains power quality should be that of a typical commercial or hospital environment. If the user of the device requires continued operation during power mains interruptions. It is recommended that the DEVICE be powered from an uninterruptible power supply or a battery.
Power frequency magnetic field EN 61000-4-8	3 A/m	NA	NA
Note: UT is the a.c. mains voltage prior to application of the test level.			

Immunity aspects at r.f.			
The DEVICE is intended for use in the electromagnetic environment specified below. The customer or the user of the DEVICE should assure that it is used in such an environment.			
Immunity test	Test Level EN 60601-1-2	Compliance level	Electromagnetic environment – guidance
RF conducted EN 61000-4-6	3 Veff from 150kHz to 80MHz	3 V/m	Portable and mobile RF communications equipment should be used no closer to any part of the device, including cables, than the recommended separation distance calculated from that equation applicable to the frequency of the transmitter. Recommended separation distance: $d = 1,2\sqrt{P}$ $d = 1,2\sqrt{P}$ 80 MHz to 800 MHz $d = 2,3\sqrt{P}$ 800 MHz to 2,5 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).
RF radiated EN 61000-4-3	3 V/m from 80MHz to 2,5GHz	3 V/m	
Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, c should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol: 			

Recommended separation distances between portable and mobile RF communications equipment and the DEVICE			
The DEVICE is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the DEVICE can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the DEVICE as recommended below, according to the maximum output power of the communications equipment.			
Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1,2\sqrt{P}$	80 MHz to 800 MHz $d = 1,2\sqrt{P}$	800 MHz to 2,5 GHz $d = 2,3\sqrt{P}$
0,01	0,12	0,12	0,23
0,1	0,38	0,38	0,73
1	1,2	1,2	2,3
10	3,8	3,8	7,3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be determined using the equation to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer
NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies. NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people. a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the DEVICE is used exceeds the applicable RF compliance level above, the DEVICE should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the DEVICE. b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

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This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.